

Bolus off Bag Instructions



Alaris® Guardrails®

A bolus can be programmed for certain approved medications* at the beginning of, or during a continuous infusion.

The bolus option appears on the Continuous Infusion page when a VTBI is entered.

Delivering a Bolus:

1. On VTBI setup page, press **BOLUS** soft key.
2. An editable starting value for bolus **DOSE** may already be entered or to enter a value, use numeric keypad. If applicable, enter patient weight.
3. An editable **DURATION** value in minutes may already be entered or to enter a time value, press **DURATION** soft key and use numeric keypad.
4. Press **START** soft key to begin bolus infusion.

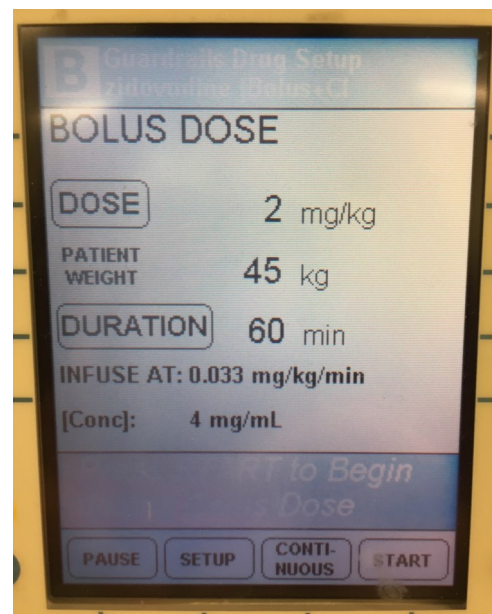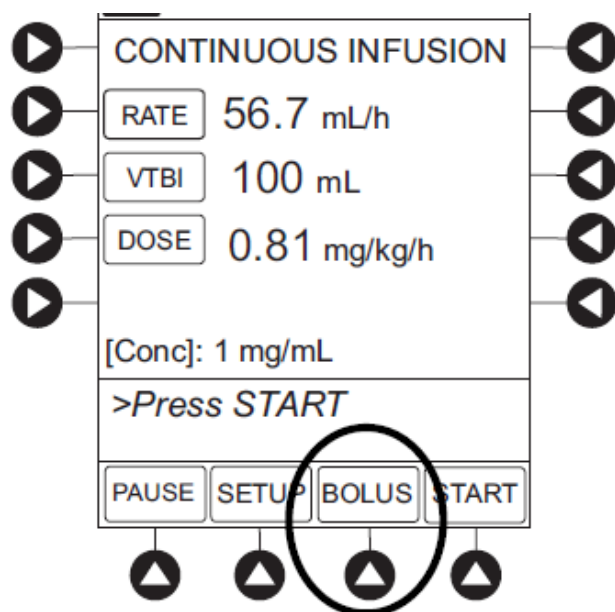

*Approved medications as of April 28th, 2018: Dexmedetomidine, DiltiazEM, Esmolol, Bivalirudin, Ketamine, Labetalol, Milrinone, Procainamide, Propofol, Zidovudine.